Pessary Directions for Use/English

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English

Incontinence Ring, Hodge and Gehrung pessaries contain metal and must be removed prior to MRI, ultrasound and x-ray procedures.

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Pessary Directions for Use / English

Cube

With / Without Drainage

Oval

With / Without Support

Dish

With / Without Support

Hodge

With / Without Support

Flexi Shelf

Ring

With / Without Support

Donut

Ring (with knob)

With / Without Support

Cup

With / Without Support

Incontinence Ring

Gellhorn

Short and Long Stem

Gehring

Shaatz

Marland

With / Without Support

PESSARY

INDICATIONS FOR USE

The flexible silicone pessary is commonly used for a mild first or second-degree uterine prolapse , proctientia or incontinence. **Pessaries are designed for single patient use and supplied non-sterile.**

CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

Pessaries are contraindicated in acute genital tract infections, pelvic infections, or noncompliant patients.

PROCEDURES

Fitting usually requires a trial of various sizes to determine the proper pessary size. Pessary Fitting Set is a valuable aid in selecting the correct pessary.

A vaginal lubricant (e.g. K-Y® jelly, Replens® or equivalent) may be required for the patient comfort.

The physician should determine if the patient requires estrogen therapy before prescribing a pessary.

1. Perform a normal pelvic examination prior to the fitting and introduction of a pessary. A pelvic exam helps determine the appropriate size. The pessary can be folded to simplify insertion and removal. The pessary should be large enough for its indication, yet not cause any undue pressure or discomfort.
2. Compress the pessary and gently insert the pessary through the introitus. Once the pessary has passed the introitus, release the compressed pessary so that it expands to its normal shape.
3. Gently move the pessary to the cervix to support the cystocele or rectocele and the uterus. When in the correct position, the physician should be able to insert an examining finger between the outer edge of the pessary and the vaginal wall. This spacing will ensure the patients comfort and reduce the risk of pressure necrosis.
4. Once in place, the pessary should not dislodge when standing, sitting, squatting or bearing down. It should not be uncomfortable for the patient during any of these routine activities.
5. When fitting a pessary, be sure to instruct the patient on its cleaning, insertion, and removal. The patient should perform these tasks before the first follow-up visit.
6. Tell the patient to report any discomfort immediately.

REMOVAL, CLEANING AND SUGGESTED FOLLOW-UP

1. To remove the pessary, carefully move the pessary forward towards the introitus. When the pessary is within reach, compress the pessary and carefully withdraw.
2. Ideally, the pessary should be removed for cleaning daily. But the removal time can be extended to weekly at the doctor's discretion. The pessary can be cleaned with mild soap and warm water.
3. Advise the patient on pessary usage during sexual activity.
4. Discuss the importance of following instructions and the expected length of time for pessary use.
5. Within 24 to 48 hours, be sure the patient is not allergic to the pessary. Examine the vagina and ask the patient if there has been any discomfort, irritation, pressure, sensitivity, or unusual vaginal discharge. Also determine if there has been any improvement in her personal symptoms.
6. Schedule follow-up visits to fit the needs of the patient.

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Wellkang Ltd