


Cervical Dilators

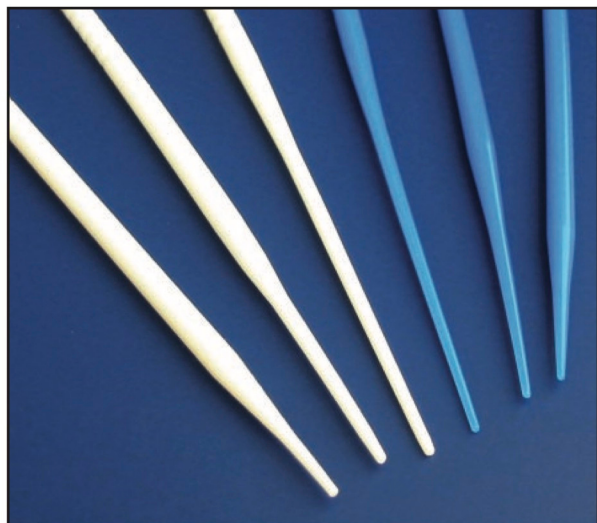

Bioteque's Cervical Dilators
(Latex-free)

Bioteque's Disposable Cervical Dilators are designed for **single-patient** use only. They are made of medical-grade polyethylene. Bioteque's Reusable Cervical Dilators may be reused after sterilizing. They are made of medical-grade Teflon.

Disposable: OS265BA-1 (OS Finder)
OS265BA-2 (Canal Finder)
OS265BA-3 (Fundus Dilator)
Reusable Set: OS260BA (OS/Canal/Fundus)

Product No.

CAUTION: Federal law requires that a physician order these devices.

PRECAUTION: Check the packaging for damage for this may compromise sterility. Also, visually inspect the dilators for any surface damage.

INDICATIONS: These Cervical Dilators are especially useful in locating and dilating the external OS, the cervical canal and the internal OS. They are particularly valuable instruments to be used on post-menopausal patients with cervical stenosis.

CONTRAINDICATIONS: Cervical dilators should not be used under the following conditions:

- Acute genital track or pelvic infections.
- A pregnancy or the suspicion of a pregnancy.

Any invasive cancer that is visible upon examination.

Any non-compliant patient.

Bioteque's Disposable and Reusable Cervical Dilators are smooth and flexible, light in weight and can be easily inserted and removed for their intended applications. Please be aware that excessive force should be avoided and these instruments should not be used for prolonged periods of time.

Each dilator is manufactured to Bioteque's exacting standards. They are flexible very smooth delivering excellent patient comfort for their intended indication.

THE OS FINDER:

A single molded band above the tactile handle and opposite the distal end of the device identifies the disposable OS Finder and the reusable OS Finder. The OS Finder may be used to find and gently dilate the external OS. Gradual dilation in combination with its tip flexibility and smooth surface will ensure maximum patient comfort. The sound of the OS Finder is tapered 23mm to an intermediate diameter of 3.8mm.

THE CANAL LOCATOR:

Two molded bands above the tactile handle identify the Canal Locator from the OS Finder and the Fundus Locator. Its sound or longer tapered tip is 45mm and extends to an intermediate diameter of 3.8mm. The Canal Locator may be used to slowly dilate the length of the cervical canal. Its flexible and smooth surface will provide maximum patient comfort.

THE FUNDUS DILATOR:

The Fundus Dilator has three molded bands opposite the distal tip that identifies this instrument from the others in the 3 piece OS Finder Set. The sound or tapered length from the distal tip is 85mm. The smooth and flexible Fundus Dilator is designed to pass through the entire length of the cervix to dilate the internal OS.