


Suction Curette (Flexible Curette)

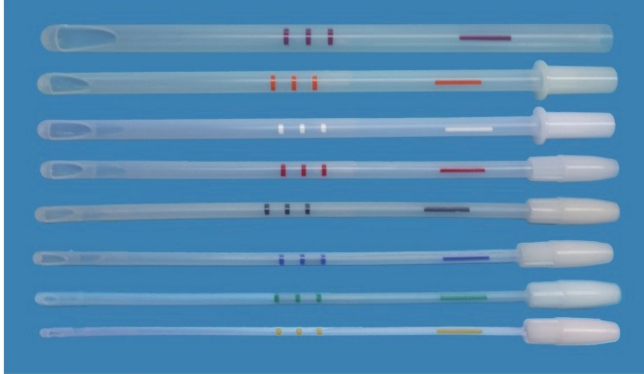

Visit our website:

www.biotequeurope.com

Tel: +3 (531) 522-7778

Suction Curette(Flexible Curette)



DESCRIPTIONS:

Flexible curette is a flexible plastic curette incorporates with vacuum aspiration syringes to provide a safe, efficient and fast vacuum aspiration method to evacuate the contents of the uterus.

It is designed for single patient use only.

SPECIFICATIONS:

Product No.	Size	Color of remark
BAFS4	4mm	Yellow
BAFS5	5mm	Green
BAFS6	6mm	Blue
BAFS7	7mm	Black
BAFS8	8mm	Red
BAFS9	9mm	White
BAFS10	10mm	Tangerine
BAFS12	12mm	Purple

INDICATIONS:

Use of Flexible Curette with the suction system for uterine evacuation and curettage in voluntary pregnancy termination or following miscarriage.

CAUTIONS:

1. Do not move the curette sideways.
2. Do not go beyond the depth measured by the hystrometer.
3. Read the instructions before using the device.
4. Use only if the package is intact.

5. Dispose after single use.

6. Use only by or under the supervision of a trained physician.

7. If the sterile package is damaged or unintentionally opened before use, do not use.

8. Store in a dark, cool, dry place.

9. Federal law restricts this device for sale to or on the order of a physician.

CONTRAINDICATIONS:

1. Patients suffering from diseases or conditions which contraindicate outpatient surgical procedures, e.g., pelvic inflammatory disease, acute infections of the cervix, clotting mechanism deficiencies, severe anemia, heart disease or extreme anxiety.

2. In general, the same criteria for regular endometrial biopsy or a D & C procedure must be followed in office curettage.

3. Contraindications of conventional D & C are contraindications for office curettage. Instructions to patients should be generally similar for both procedures.

DIRECTIONS FOR USE:

1. Measure uterine depth.
2. Dilate the cervix if necessary.
3. Introduce the approximately size of Flexible Curette through the cervical canal into the uterus up to the fundus.
4. Connect the Flexible Curette to the suction system.
5. Operate the procedure by rotating the curette 360° to

aspirate throughout the uterus cavity accurately.
Repeat the operation two or three times.
6. Withdraw the curette from the uterus while continuing the rotating movement.

WARNINGS:

Dispose of medical waste properly in accordance with regulations after using.

PRODUCTION:

Each Flexible Curette is sterile and packaged individually. The curette should be discarded after a single use.

PRINCIPLE:

The diameter of the curette to be used corresponds to the number of weeks of pregnancy.

Approximate uterine size (weeks LMP*)	Approximate KC size
5-7 LMP	5mm
7-9 LMP	6mm
9-12 LMP	7-12mm

*LMP=Last Menstrual Period



Not made with natural rubber latex



Do not reuse



Batch code



Expiry date



Upper limit of temperature



Consult instructions for use



Sterile EO